



Manual: Assembly of Bottle Holder on Walker 12er size S




① Place of assembly of the bottle holder (R or L).



② Wrap the strap around the frame tube and pull it through the holder.



③ Pull the strap tight and turn the nut clockwise to tighten the strap.

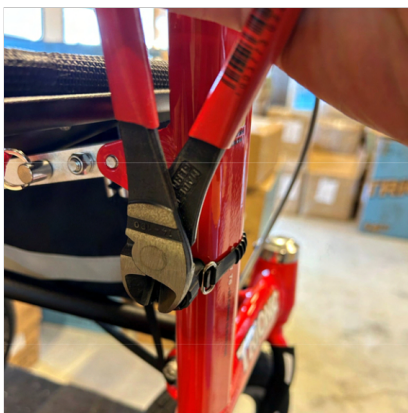

④ Cut the excessive strap off.



⑤ Mount the second holder in the same way.



⑥ The two holders correctly assembled on the frame tube.



⑦ Attach the bottle holder with the two screws. The "opening" of the bottle holder should face outwards.



⑧ The bottle holder is now attached to the frame and you enter the bottle from the side and remove it outwards.