




## Comfort Folding Stool 75XXL

3-legged Comfort Folding Stool - Seat height 75cm/30" – Compact, lightweight and easy to bring along when heading outdoors - Including carry bag - Max load 250 kg

Scan the QR-code with your smart phone or device to view video instructions.

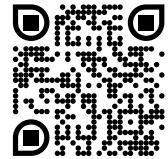

- The Comfort Folding Stool follows you wherever you go.
- It is compact enough to carry in your bag, and it simply folds away.
- You can quickly extend the legs and fold it out, and it's stable just about anywhere.
- Comfortable to sit on – for a well-deserved rest on your daily outdoor walk, when gardening, fishing or when waiting in that never-ending queue.
- The Comfort Folding Stool helps to protect your back, knees and joints.
- The Comfort Folding Stool is available in different heights with different seat sizes; 55XL, 65XXL and 75XXL.



- ① Unfold the stool.
- ② Pull out the legs and ensure that the red lock buttons are in the correct position and that they are visible after extending the telescopic legs.
- ③ The Comfort Folding Stool is now ready to be used at its normal seat height.
- ④ To fold the stool after use, press the red button and push the telescopic leg back into its non-extended position and fold the stool together.



- ⑤ The Comfort Folding Stool offers two seat heights. When for example gardening at low heights, sit on the Comfort Folding Stool without extending the telescopic legs.



- ⑥ To lock the lower legs when the Comfort Folding Stool is folded together, twist the legs so they are locked by the rubber feet being close together. To use the Comfort Folding Stool, do the opposite and you can unlock the lower legs.



- ⑦ It can easily be folded together to fit in its carry bag – ready for the next adventure.

### ***If You have the "Steady" stability straps for the Comfort Folding Stool***



The Steady stability straps increase the safety and stability of the Comfort Folding Stool on soft surfaces like sand or soft soil.

- ① The three attachment points of the Steady can be moved to fit the size of your Comfort Folding Stool.
- ② Adjust the attachment points to their correct position.
- ③ Insert the feet of the stool into the pockets of the Steady.



Improper use of the Comfort folding stool can result in bodily injury or harm. Always check the Comfort Folding Stool before using it to make sure that the red lock button is in the correct position.

Be careful when sitting on the Comfort Folding Stool on uneven ground. Avoid putting pressure on only one or two legs of the Comfort Folding Stool when seated.

***The Comfort Folding Stool is a Swedish invention and the only 3-legged stool in the world with telescopic legs, patents and trademark protections. It is light, compact and offers a superior seating comfort.***

#### **Manufacturer :**

Scandinavian Touch AB  
Lindhagsgatan 5C  
531 40 Lidköping  
SWEDEN  
+4651025555